



















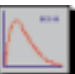







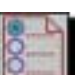
















## ... frequently used buttons in the COMPACT

- |   |   |
|---|---|
|  ...Return to the home screen  |  ...Go to the next screen or logical step and save changes         |
|  ...Access the help files  |  ...Cancel a selection   |
|  ...Recalls an existing subject from the database  |  ...Perform a Post test  |
|  ...Enter the details of a new subject   |  ...Perform a Single Breath Test                                   |
|  ...Perform a test session   |  ...Perform a CCS Test   |
|  ...Print a test session, calibration log or report  |  ...View suggested Interpretation of the session results           |
|  ...Access additional COMPACT functions   |  ...Recall an existing test session stored in the database        |
|   .....Scroll through screens |  ...Incentive device used in the single breathe test of children |
|  ...Access the configuration options   |  ...Enter comments/text  |
|  ...Return to previous screen without saving changes   |   |

## ... frequently used icons in the COMPACT

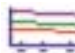- |   |   |
|---|---|
|  .....Unit is ready to accept a blow |  .....Smoker   |
|  .....Subject's name                 |  .....Ex-smoker  |
|  .....Subject's first name           |  .....Cigs. per day                                    |
|  .....Subject's last name            |  .....Year started smoking                             |
|  .....Subject I.D.                   |  .....Year quit smoking                                |
|  .....Population group               |  .....Graph tab when performing single breath test     |
|  .....Date of birth                  |  .....Graph tab for Closed Circuit test (Flow/Volume)  |
|  .....Male                          |  .....Graph tab for Closed Circuit test (Volume/Time) |
|  .....Female                       |   |
|  .....Height                       |  .....Test session results tab                       |
|  .....Weight                       |  .....Subject history trend tab                      |
|  .....Non-smoker                   |  .....Test session information tab                   |